



Dear User,

Congratulations on your choice and purchase of the VISOR180 mask. We believe it will help you in your everyday tasks with less exposure to harmful externalities. In parallel, due to the mask, you are also less likely to expose your surroundings. The mask is a Polish product, manufactured in accordance with EU norms and regulations relevant for this type of protective equipment. As an experienced and recognized producer we assure that VISOR180 mask is manufactured according to the high hygiene and quality standards, confirmed by the BRC certificate. The VISOR180 mask is made of high quality, flexible, non-cracking polyester (PET) - very lightweight and transparent fabric. It can be disinfected with standard disinfectants (even containing up to 95% alcohol). Remember, in order to keep your mask highly transparent, preferably apply disinfectants that do not require wiping or use a soft cloth when cleaning. We care for the natural environment, therefore our mask is easy to recycle - even 100%, if you don't need to put or will remove the foam bar before discarding.

We wish healthy and pleasant use.

We are constantly developing our products, so should you have any remarks or comments please share with us. You can do that via WWW.VISOR180.EU

VISOR180 Face Shield (Mask) Manual

(Tutorial video is available on the website www.visor180.eu)

I Preparation:

1. Wash / disinfect your hands to avoid leaving fingerprints and other marks on the mask. Then place all the parts of the kit in front of you: mask, strap, name tab, foam bar (fig. A). If you do not have a clean surface (e.g. fold it on the floor), you can use the cardboard insert from the packaging as a pad.
2. Sign the name tab - it can be, for example, your name, function, greeting or favorite slogan (Fig. B).


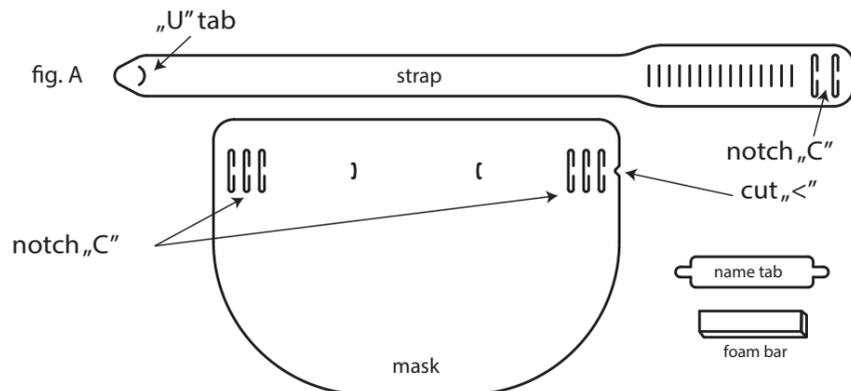


fig. B

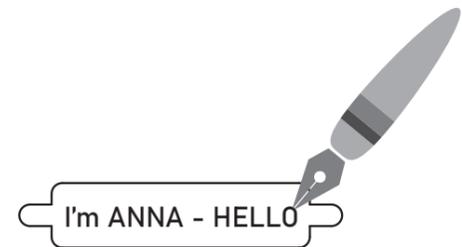

II Assembly:

3. Place the mask in front of you, so that the cut "<" on the side is on your right (fig. A)
4. Starting from the right side, put the narrow end of the strap through the three notches "C" on the mask starting from the bottom side (Fig. C).
5. Pull the strap and then put it through the three "C" notches on the other side of the mask.
6. Reverse the mask and place the name tab in the notches / holders in the center of the mask (Fig. D).
7. Reverse the mask again so that the narrow end of the strap is back on your left side.



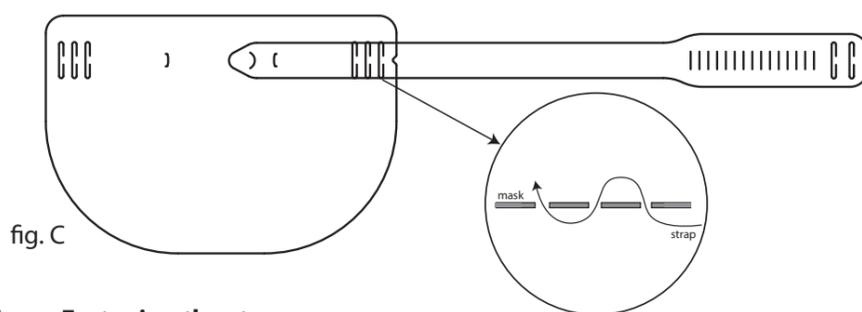


fig. D

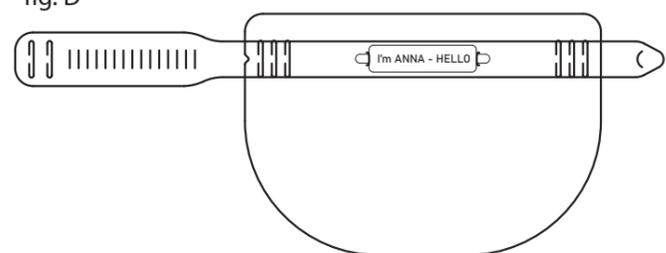

III Fastening the strap:

8. Stretch the strap, curving the mask.
9. Put the narrow end of the strap through the two notches "C" of the wider strap end, STARTING FROM THE TOP (fig. E).
10. Pre-adjust the strap length and diameter to your head.
11. Press the "U" tab of the narrow end of a strip into the desired notch "I" of the wider strip end. Lock the very end of the narrow end of a strap in the one of the following "I" notches.

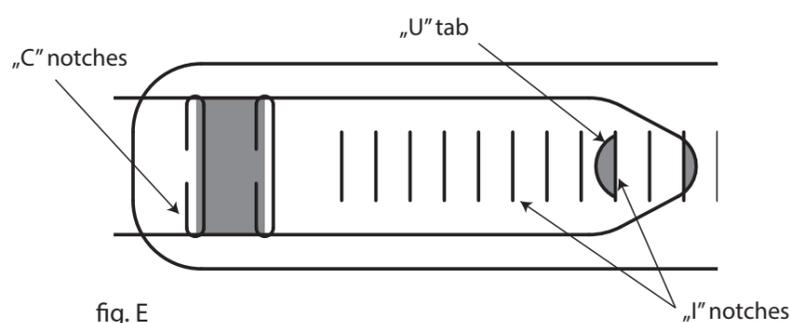

IV Adjusting the mask

12. Adjust the mask bow (e.g. slide it out if you wear glasses).
13. Decide if you need to apply a foam bar at the height of your forehead for greater comfort. Please remember to remove the foam bar and the name tab before discarding used mask it into the trash bin for "PLASTIC". Foam bar should be disposed to the "OTHERS" trash bin, and the name tab to the "PAPER" (Fig. F).

fig. F

